


DAILY PATIENTS PAGE

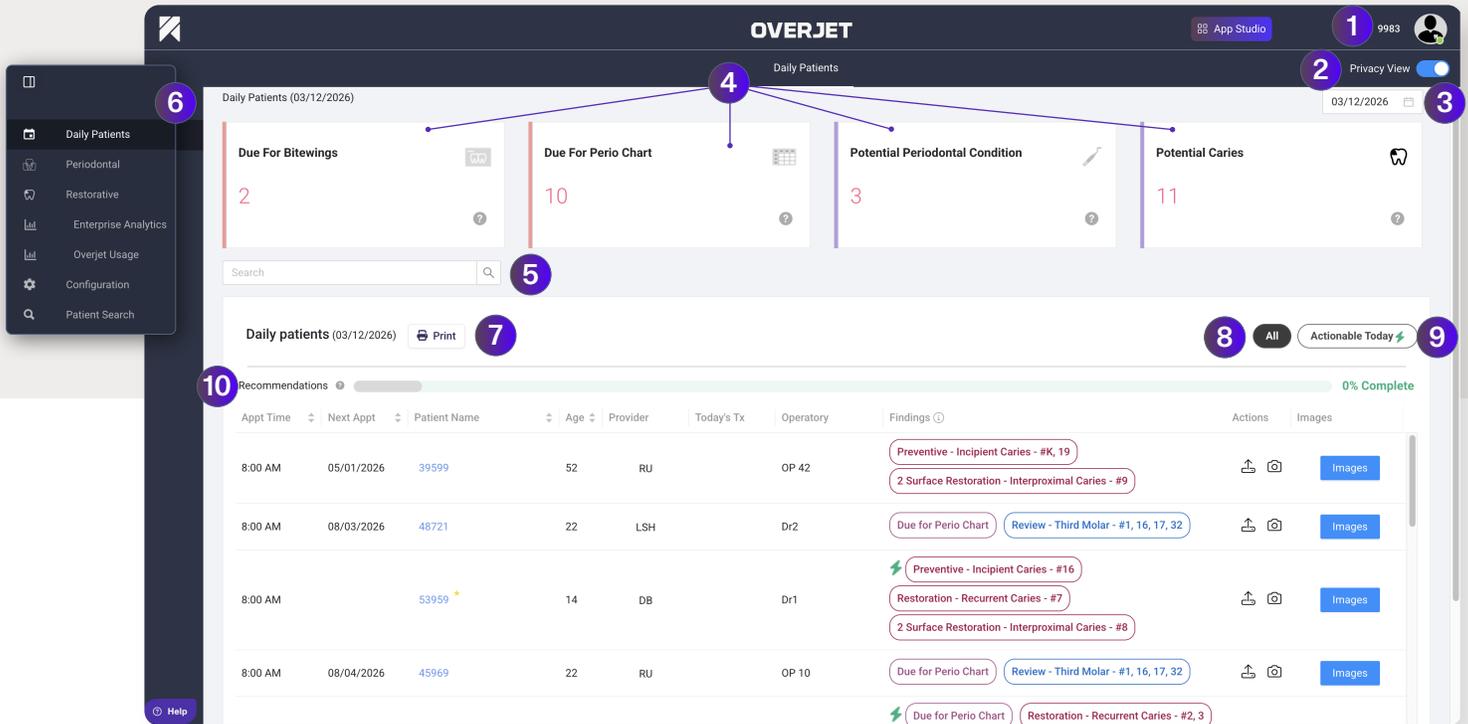

1 **Clinic ID:** If you have access to multiple locations, click the silhouette and the options will show in a dropdown menu.

2 **Privacy View:** Toggle to change the patient names to patient ID numbers to protect PHI.

3 **Calendar:** This defaults to today's date. You may select other dates to view by clicking on the desired date.

4 **Filter Cards:** These cards can be used to filter the Daily Patients page for specific criteria.

5 **Search Bar:** Type in the operatory name or provider name to filter the list by the desired criteria. You may also search for a patient name or patient ID on today's schedule.

6 **Left Navigation Bar:** Hover over the icons to see the names for each one.


- Daily Patients Page
- Periodontal
- Restorative
- Configuration
- Patient Search

7 **Print button:** Click this icon to print the Daily Patients page.

8 **All:** A list of all Overjet findings.

9 **Actionable Today:** Click to view a list of potential findings for your consideration that Overjet has identified as being able to address at today's appointment.

10 **Recommendations:** This green bar shows the percentage of recommended Overjet findings that have been completed

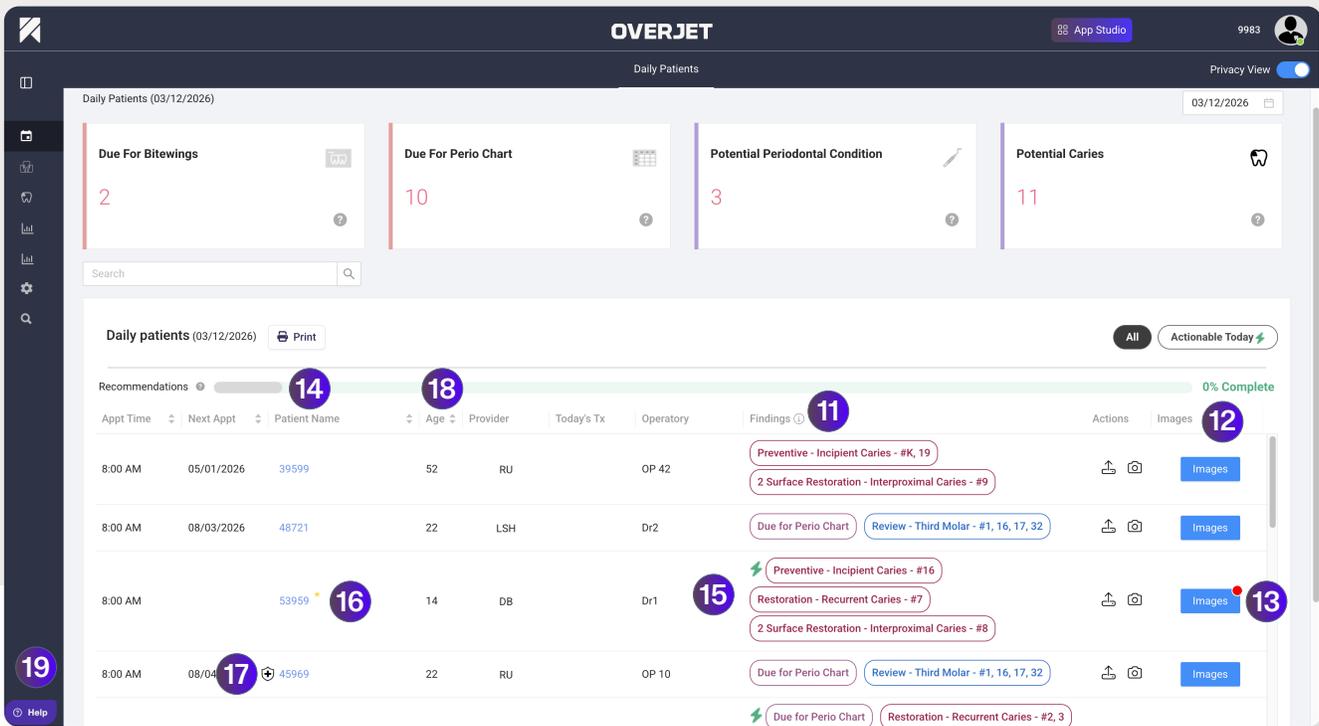

11 Findings column: A list of potential treatment needs that have not been diagnosed yet and of patients who may be due for periodontal probing, radiographs, or sealants.

- Findings that are completed will be automatically updated as Overjet syncs with the digital imaging system and practice management system.
- To track why a finding was not completed, click on the finding to bring up a list of reasons and select the most appropriate. The list of reasons changes based on the type of procedure.

12 Images: Click this button to access the patient's radiographs. New images taken today will appear within a few minutes after saving them in your imaging software.

13 Red dot: Signifies that radiographs taken today are ready, but have not been looked at yet from this workstation.

14 Patient Name: Click the patient's name or Patient ID (if in privacy view) to pull up that patient's transaction history.

15 Findings Legend:

- Green: Indicates the finding has been treatment planned or completed.
- Purple: Indicates the patient may be due for radiographs, periodontal charting, or sealants. Red: Indicates AI findings of potential restorative needs.
- Blue: Indicates AI findings of potential periodontal needs.
- Yellow: Indicates some AI findings have been treatment planned or completed, but not all. Gray: Indicates that the AI finding was rejected or noted. *Right-click to reject or note. New: Indicates the finding is from an image taken today.
- Lightning Bolt: Indicates that at least one of the findings is able to be completed today.

16 Gold Star: This indicates the patient is a new patient.

17 Icons: Symbols to the left of a patient's name indicate the type of insurance.

18 Age: You can view images of patients under 4 years old, but these images won't show any findings.

19 Help: Open the help widget to submit issues and questions to our support team.