



OVERJET

Platform Legend

Dashboard

Patient Name: Patient names are clickable for a review of treatment history



Icons - Symbols to the left of a patient's name indicating type of insurance



Gold Star - Located to the right of a patient's name and indicates a new patient

Radiographs:



Red Dot - Signifies that radiographs taken today are ready for review in Overjet

Findings:



Maroon Outline - Indicates the patient is due for Bitewings or Perio Chart



Red Outline- Indicates AI findings of potential restorative needs (not yet treatment planned)



Blue Outline - Indicates AI findings of potential periodontal needs (not yet treatment planned)



Green Filled - Indicates the finding has been treatment planned or completed



Yellow Filled - Indicates some AI findings have been treatment planned or completed, but not all



Gray Filled- Indicates that the AI finding was rejected or noted- *Right-click to reject or note

NEW

New - Indicates the finding is from an image captured today



Lightning bolt - Indicates that at least one of the findings is able to be completed today

Help:



Help Widget - This button allows you to quickly contact Overjet for technical support

FMX Page

From the dashboard, click on the blue images button to view the patient's FMX page

*This will take you to a mounted view of your patient's most recent radiographs



*Click on any image to view it in full screen on the canvas page

Timeline: The timeline at the top of the page has clickable circles-Click the circles to view images from earlier dates



Indicators: Colorful dots may be present above images. These indicate specific findings within the image below.

- Caries Present
- Perio Present
- Restoration Present
- Third Molar Present
- PARL Present

Additional Features:



Use the Export button in the upper left to download selected images



Click "All Recent Radiographs" in the upper right to view images from multiple dates in one view



Click "Exit to Daily Patients" to return to your dashboard

Canvas Page

Findings/Visualizations Panel:

-  **Eyeball Icon** - Hides finding/visualization from image; adds finding/visualization to image
-  **Trashcan** - Removes individual findings/visualizations

Colorful Findings:

-  **Red** - Indicates decay through the DEJ
-  **Orange** - Indicates decay not yet through the DEJ, incipient lesion
-  **Blue** - Existing restorations
- White Box** - Marginal discrepancy
-  **Orange Box** - Calculus detection
- Bone Level Measurements/Ratios** - Indicate bone loss from CEJ to crest of bone
 -  **Green Measurements** - 0.0mm-1.9mm on BWs, 0%-14% on PAs
 -  **Yellow Measurements** - 2.0mm-2.9mm on BWs, 15%-32% on PAs
 -  **Red Measurements** - 3mm + on BWs, 33% + on PAs
-  **Bright Yellow** - Indicates PARL

***Reminder - The spacebar will turn on/off all findings and visualizations. Letter "A" will turn on/off anatomical landmarks**

Canvas Page

Tools to Adjust Image:



Image Adjustment Icon - Click the arrow to turn it on (green)- this allows you to control brightness and contrast using your trackpad

- Moving the cursor to the left or right adjusts contrast.
- Moving the cursor up or down adjusts brightness.
- Moving the cursor diagonally adjusts both brightness and contrast.

Rotate Icon - Each click on this tool will rotate the image clockwise 90-degrees.

- Once image has been rotated to desired position, click “Run AI” to save adjustment and rerun image through the logic.
- To discard changes, click “Discard” button.

Flip Icon - Horizontally or vertically flip your image.

- Once the image has been flipped to the desired position, click “Run AI” to save adjustment and rerun image through the logic.
- To discard changes, click “Discard” button.

Spotlight Icon - Click image to magnify specific areas. Magnified area will move on image as mouse is moved.

Magnifier Icon - Click image to magnify specific area. Magnified area does not move as mouse is moved.

Brightness Icon - Use the slide bar to adjust image brightness.

Contrast Icon - Use the slide bar to adjust image contrast.

Gamma Icon - Use the slide bar to adjust gamma (helpful for images that are too bleached out or too dark)

Auto Enhancement Icon - Green icon indicates automatic enhancement of image brightness, contrast and sharpness. Turn off auto enhancement by clicking icon.

Reset Zoom Icon - Reset image to the original size after zooming in or out.

Add Icon - Add orange boxes to your image to indicate calculus.

- To save changes, click “Save” button.
- To Discard box, click “Discard” button.

Reset Image Adjustments Icon - Will appear after an image has been altered. Click icon to reset image to default.

 **OVERJET**